

**WARNING:** When reassembling the wheelchair after removing the rear wheels, insure that the rear axles are locked in place to prevent wheels from falling off.

### Street Use

Your wheelchair should not be operated on roads and streets except when using crosswalks. At such times, use only marked pedestrian crosswalks and proceed with extreme caution.

### Transferring

Never stand on the footrest footplates while transferring because the chair may tip over. Always engage the wheel locks before transferring in or out of your wheelchair.

### Wheel Locks

Wheel locks are designed to prevent the rear wheels from turning when the chair is stopped. It is not intended to slow down the chair when it is moving. Adjust the wheel locks after making any changes to the rear axle sleeves position.

### Weight Limit

Maximum user weight is 200 lbs for KM-TV10.

**WARNING:** This wheelchair has not been approved as a seat in a motor vehicle. Always transfer to an approved vehicle seat.

**WARNING:** Do not tie down or attach anything to the wheels in any way. This could cause tipping and possibly result in injury or damage to the chair.

**WARNING:** Attempting inclines steeper than 6 degrees can cause your chair to tip over. Use of anti-tipping device is highly recommended for inclines.

**WARNING:** Do not operate your chair on hilly or rough terrain or in conditions beyond your ability. They could cause you to lose control of your chair and should be avoided.

**WARNING:** Engage wheel locks when using this wheelchair in an elevator or wheelchair lift, and when transferring.

**WARNING:** Before transferring to or from wheelchair, place wheelchair on a stable, level surface and engage wheel locks.

## SPECIFICATIONS



- |                                      |                                    |
|--------------------------------------|------------------------------------|
| 1. Push Handle Backrest Tube         | 8. Rear Wheel Lock Brake           |
| 2. Padded Folding Armrests           | 9. Rear Wheel Lock Brake Handle    |
| 3. Backrest                          | 10. Folding Chain (Push/Pull Down) |
| 4. Padded Seat                       |                                    |
| 5. Flip-up Footrest                  |                                    |
| 6. Front Caster w/ Polyurethane Tire |                                    |
| 7. Rear Caster w/ Polyurethane Tire  |                                    |

## OPERATING

### **1. Opening Travel chair**

Lay KM-TV10 travel chair on flat surface toward you on the side (As figure.1) Make sure both the front and rear casters DO NOT align (see figure.2 ) Lock rear wheel lock brake. Hold Point A (at the edge of the padded seat) then Pull at Point B (push-handle backrest tube) until fully extend in the upright position. Seat the user down on KM-TV10, then fold down armrest and footrest. Unlock rear wheel lock brake.

**WARNING:** When unfolding make sure the front casters and the rear casters DO NOT align (see figure.2 )



**Figure. 1 CORRECT**



**Figure. 2 WARNING: WRONG**

### **2. Folding Travel chair**

Lock rear wheel lock brake. Fold up armrests and footrests, then unseat user from KM-TV10 before folding the chair. Hold Point B (push-handle backrest tube) with one hand, then while push/pull down folding chain with the other hand - - fold the backrest until flat.

## GENERAL MAINTENANCE RECOMMENDATIONS

For maximum service, it is recommended that the wheel bearings be cleaned, inspected and regreased about once a year by dealer. Adjust brakes when necessary. Check periodically for loose screws, bolts or spokes and repair immediately.

The best way to preserve the finish is to keep it clean by sponging with a damp cloth and drying immediately with a soft dry cloth. Most of the hinged or movable parts may be lubricated with “White Graphite”.

## SAFETY TIPS

**WARNING:** Never stand on the footrests, for the chair may tip forward.

### **Opening and Folding the wheelchair**

Do not place your fingers between the folding mechanism when opening or folding the wheelchair.

**WARNING:** When unfolding make sure the front casters and the rear casters DO NOT align (see figure.2 ) or damage to folding mechanism will result if extreme force is applied.

## WARRANTY

One year warranty on main frame and side frame, six months on parts.

