

User's easy Manual

Stop

Press STOP to halt any operation. Hold STOP for 3 seconds to turn off unit.

Bidet

Feminine wash(front)
Runs 2 minutes unless you press STOP

Power Wash

To ease constipation.
Runs 2 minutes unless you press STOP

Dry

Will run total of 3 minutes unless you press STOP

HI / LO

Adjusts water pressure on WASH/BIDET/POWER
Adjusts air temperature on DRY

Massage

Nozzle moves forward and back intermittently. Adjustable stream width. Can also be used in Child mode.



Massage Control


Massage width is adjusted during massage operation. (DIB-2300/2500 can be used with Child Use)

Seat Temp

Adjusts temp of seat
(93°/100°/107°F)
(34°/38°/42°C)

Water Temp

Adjusts temperature
(93°/96°/100°F)
(34°/36°/38°C)

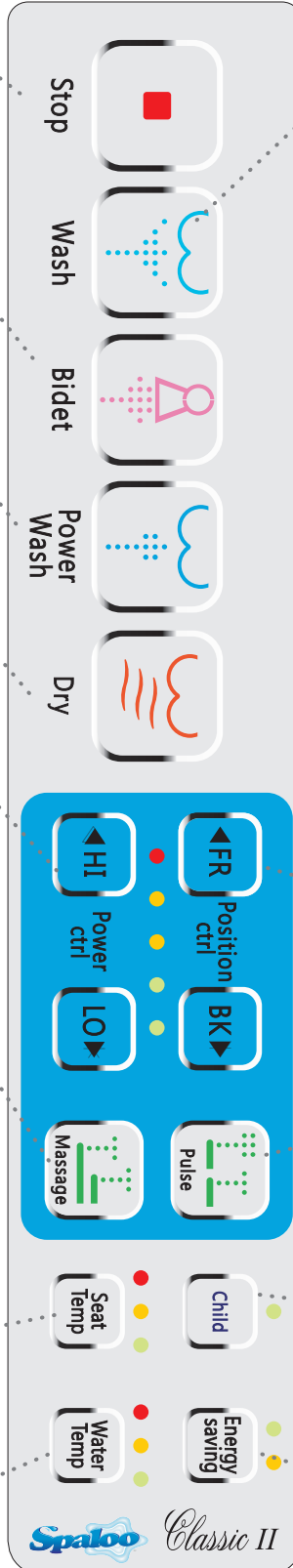

Wash

Washes posterior(back).
Runs 2 minutes unless you press STOP

Cleaning of nozzle

1. Press the Wash button for 3 seconds while not seated, and then the nozzle will move forward and stop. (for 2 minutes)
2. Clean the nozzle with a unused toothbrush, etc., and press the Stop button, then it will return to the original place for self cleaning.

Deodorizing fan speed

While pressing the STOP button, press the Water Temp button.

Water Temp (LED)			Speed of Deodorizing Fan	
Red	Yellow	Green	Seated	Unseated
OFF	OFF	OFF	OFF	OFF
OFF	OFF	ON	Low speed	Low speed
OFF	ON	ON	Low speed	High speed
ON	ON	ON	High speed	High speed

FR / BK

Front/Back Adjusts nozzle location in Wash /Bidet/Power function

Pulse

Pulsating water stream (Strong/Weak) Use with Wash/Bidet/Enema Wash

Child

Safety Mode;
Nozzle moves forward;
water pressure reduced.

Energy Saving

- 1st. Automatic mode
- 2nd. Learn Energy saving Mode
- 3rd. Cancel