


ASSEMBLY INSTRUCTIONS

Caution: Need at least two persons to complete the assembly

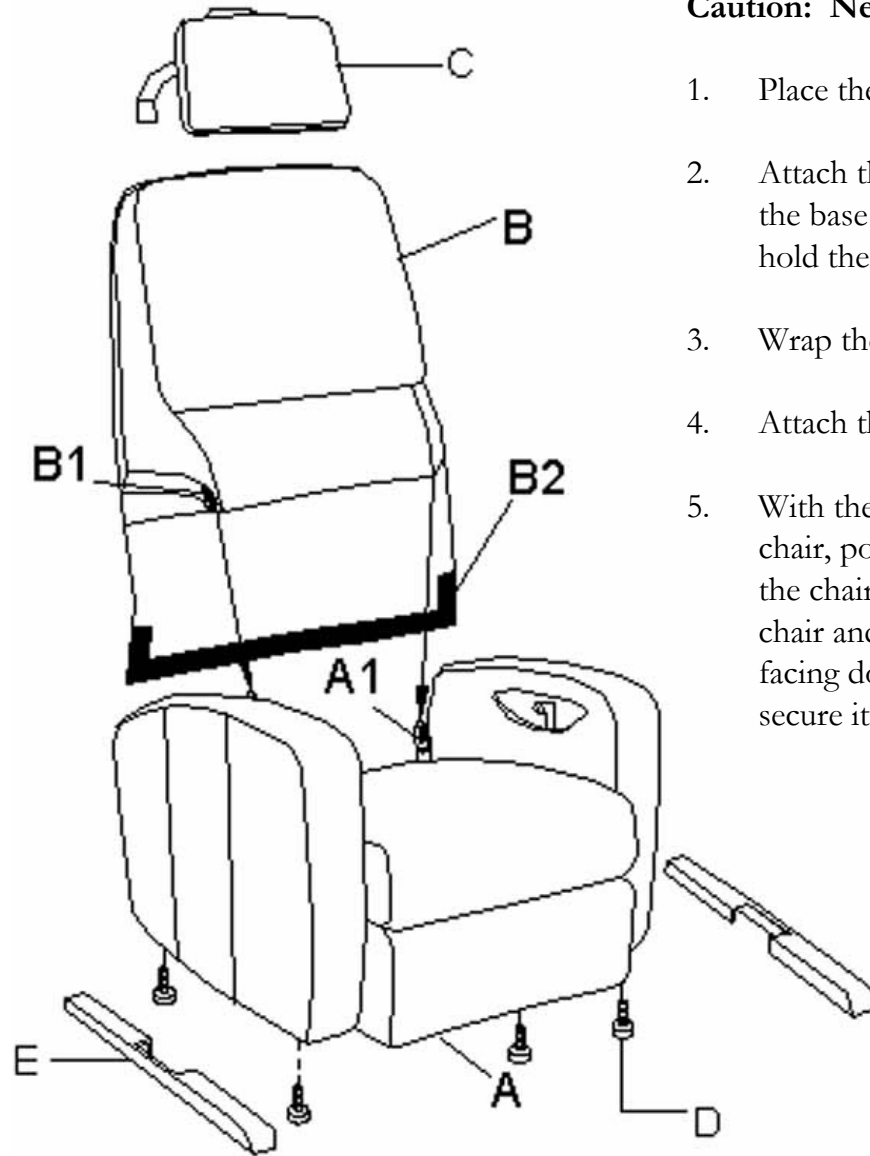

1. Place the base (**A**) in the desired location in room
2. Attach the back (**B**) to the base (**A**) by guiding the metal prongs (**A1**) of the base into the metal channels (**B1**) inside the back. One person should hold the back while the other guides the prongs into the channels
3. Wrap the velcro (**B2**) around the bar located at the rear of the base
4. Attach the pillow (**C**) to the back (**B**)
5. With the long end of the wooden leg cover (**E**) toward the rear of the chair, position the cover sideways with the open side facing inward toward the chair. Then, place the leg cover directly against the metal leg of the chair and rotate the cover into an upright position (with the open side facing down toward the floor). Gently press down on the leg cover to secure it into place

PARTS LIST

A.	Base	1
B.	Back	1
C.	Pillow	1
D.	Feet	4
E.	Wooden Leg Covers (L&R)	2