

When resident falls occur...

- ✓ Reduce risk
- ✓ Protect employees
- ✓ Save costs



The modern, dignified
solution for lifting
residents who have fallen

The **CAMEL**
Complete Air Moving Elevating Lift

A Resident has fallen...



Step 1

- After determining the person is not hurt, place them on their side in the rescue position.
- Then log roll (or slide) the person onto the deflated **CAMEL**.

Step 2

- Elevate back and head section until person is comfortable, then start inflating the **CAMEL** from the bottom up.

Reduce risk, protect employees, save costs



Step 3

- Continue to inflate the **CAMEL**, level by level until the resident is in a position where they can stand up.

Step 4

- After the **CAMEL** has been properly inflated, assist the resident to stand up, transfer to wheelchair, or use a walker to ambulate.



***The CAMEL lifting system includes:
2 Airflo compressors, lifting cushion,
carrying bag, slide sheet, and transfer belt.***

Camel by Mangar

Easy controls!



Simple, color coded connections



Slide sheets make it easy to help immobile patients



The Mangar transfer belt helps secure patients





Airflo

The versatile Mangar Airflo compressor is engineered to operate Mangar's entire range of air-powered lifting products and conforms to the industry's highest safety standards.

One compressor for many products:



Benefits

- Easy to use — Simple, color coded connections
- Quiet operation
- Portability
- Normal or high flow option
- Auto function



Mangar USA evolved to become a globally focused company with an extensive range of award winning and internationally acclaimed products that provide solutions to a wide variety of lifting, moving and handling problems.

At Mangar USA we maintain the highest standard of customer service with integrity.

The CAMEL

Complete Air Moving Elevating Lift

Properly inflated the CAMEL allows residents to stand, or transfer, to a wheelchair or walker in minutes reducing the risk of lifting injuries!



- Particularly suitable for very large or bariatric people (lifts up to 700lbs!)
- **Made of durable NYLON, coated in polyurethane**
- Has unique internal structure to increase strength and rigidity
- Provides secure, rigid support for the person being lifted
- Portable
- Packs away into a compact and discrete bag
- For use indoors or out
- Minimizes manual handling risk - great for zero lifting policies
- Includes 2 Airflo battery operated compressors

Product Specifications

Max user weight		700lbs
Inflated dimensions	width	28"
	seat depth	18.5"
	total length	44.5"
	height	22"
Product weight		14lbs

Call us today for a complete demonstration at 800-901-5269.

Or view our product videos online at www.mangarusa.com.

