Another Genuine Big John Product " wide from Side to Side

Big John Products 8533 Canoga Ave. Suite D, Canoga Park, CA 91304 **TOLL FREE: 866-366-0669**

ADA Compliant Registration Number 4.23 ADAAG

Meets or exceeds


ASME / ANSI Z124.5

CSA

IAPMO / UPC



MADE IN THE U.S.A.

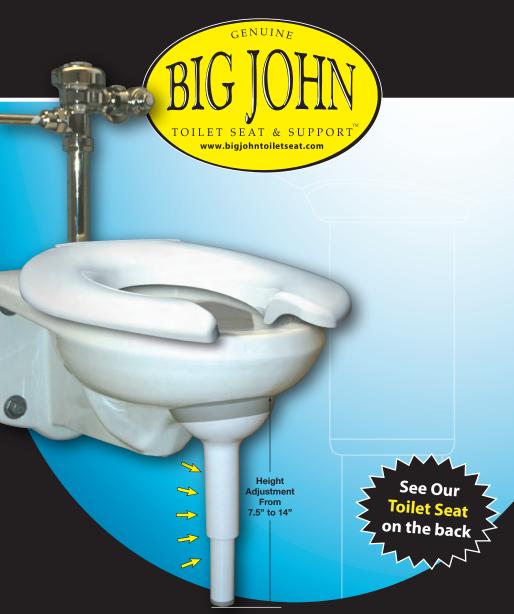

Installation Instructions

MADE IN THE U.S.A.

www.bigjohntoiletseat.com

6 Easy Step-by-Step Instructions for Installing the Big John Toilet Support

Step 1

For toilets that have a raised section at the bottom of the toilet bowl, remove the enclosed insert to allow for a tight fit.

For toilets that don't have a raised section at the bottom of the toilet bowl, put the insert into the grooves on the carriage.

Step 2

Remove liner from adhesive tape on the top of the carriage and place the *Toilet Support* in position under the toilet bowl.

DO NOT CAULK THE TOP OF THE CARRIAGE. MAKE SURE BOTTOM OF TOILET IS CLEAN BEFORE INSTALLING. Step 3

With the *Toilet Support* under the bowl unscrew the leg to tighten it snugly against the floor.

DO NOT OVERTIGHTEN.

Step 4

Make sure the *Toilet Support* is mounted straight and that it is centered under the toilet.

Step 5

Wipe around the entire carriage and toilet where they meet, with the enclosed alcohol wipes, to remove any dirt.

Step 6

Apply a bead of the enclosed caulk around the entire *Toilet Support* carriage where it meets the toilet. Make sure to fill in all gaps.

Wipe off any excess with a damp cloth. Allow it to cure for 24 hours.

Once caulk has cured, leg can be raised to clean underneath it. Re-lower the leg after cleaning, making sure that it fits snugly.



TOLL FREE: 866-366-0669 www.bigjohntoiletseat.com