


The
Adjustable Advantage™
naturally better by design *Toilet Seat*

Installation Instructions

Fits Round and Elongated Toilet Bowls

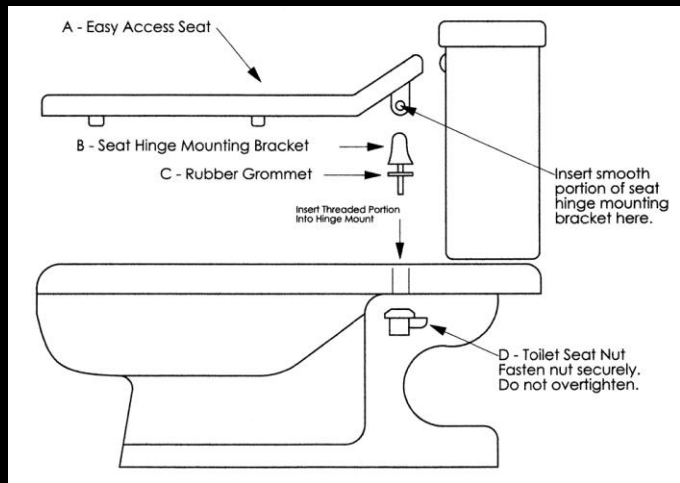

1. Remove and discard old toilet seat.
2. Clean all surfaces between lower and upper hinge bracket mounting holes.
3. Insert smooth portion of hinge mounting bracket on each side of seat hinge holes.
4. Guide rubber grommets onto each threaded portion of hinge mounting bracket.
5. Guide each threaded portion of hinge bracket onto the toilet seat mounting holes.
6. Attach each nylon toilet seat nut to threaded portion of hinge mounting bracket.
7. Tighten each nut thoroughly making sure not to over tighten or strip nut thread.
8. Check seat for excess movement and tighten nuts if necessary.

The **Adjustable Advantage™**

Seat leg supports that expand up to 6" to the left and right
Adds up to 12" of additional access for improved hygiene
Adjusts for greater comfort, support, relaxation and health

AdjustableAdvantage.com