

PostureLITE™

BY ROHO®

ENJOY A
NEW LEVEL
OF COMFORTABLE SKIN
PROTECTION!
Appréciez un nouveau niveau
de protection pour la peau
confortable!

WHEELCHAIR SUPPORT SURFACE DESIGNED TO PROTECT YOUR SKIN WHILE SEATED

Surface de soutien pour fauteuil roulant conçue
pour protéger votre peau, lorsque vous êtes assis.



SELF-ADJUSTING FOAM CUSHION ENCASED IN AIR BLADDER WITH AIR EXPANSION CHAMBERS

Coussin de mousse auto-réglable emballé dans un
ballon d'air avec des chambres d'expansion d'air

CONFORMS TO YOUR BODY TO PROVIDE BASIC SKIN PROTECTION

Épouse les formes de votre corps
afin d'offrir une protection
de base à votre peau

STRONG, DURABLE CONSTRUCTION

De fabrication
robuste et durable

EASY FIT SIZES

Tailles qui
conviennent
à tous



Comfortable protection to meet your needs.
Innovations in Seating Technology and Materials. By ROHO®.
Une protection confortable afin de répondre à vos besoins.
Innovations en technologie d'assise et matériaux. Par ROHO®.







ENGLISH	2 - 3
ESPAÑOL	4 - 5
FRANÇAIS	6 - 7
DEUTSCH	8 - 9
ITALIANO	10 - 11
NEDERLANDS	12 - 13
DANSK	14 - 15
SVENSKA	16 - 17
NORSK	18 - 19
SUOMI	20 - 21
PORTUGUÊS	22 - 23



PostureLITE™ Cushion by ROHO® User Manual

SUPPLIER: This manual must be given to the user of this product.

USER: Before using this product, read these instructions and save for future reference.

ROHO, Inc. has a policy of continual product improvement and reserves the right to amend this document. Current version of this document is available on the website, www.therohogroup.com.

INTENDED USE:

The PostureLITE™ cushion by ROHO® (PostureLITE) is a self-adjusting air and foam wheelchair support surface. The PostureLITE is intended for individuals who weigh 350 lbs. (158 kg) or less and who may require skin protection while seated. Cushion insert must be used with supplied cover.

! CAUTIONS

- **PUNCTURE:** Keep cushion away from sharp objects which may puncture the cushion insert and cause deflation.
- **CUSHION AND COVER ORIENTATION:** PostureLITE cushion insert must be used with cover provided, and oriented following product labeling.
- **INSTABILITY AND FALL HAZARD: DO NOT** use cushion on top of, or in conjunction with, any other products or materials, except as indicated in INTENDED USE. Doing so may add to user instability resulting in falls. Consult with your clinician for proper usage.
- **DO NOT** expose cushion to high heat, open flames, or hot ashes.
- **CLIMATE:** Avoid exposing cushion to extreme temperatures when not in use.
- **OBSTRUCTIONS: DO NOT** place any obstructions between user and cushion, as it will reduce product effectiveness.
- **DO NOT** use cushion as a water flotation device (e.g. a Life Preserver).
- **OZONE GENERATORS:** Prolonged exposure to ozone will degrade materials used in the PostureLITE and may affect the performance of your product and invalidate the product warranty. Avoid prolonged exposure of the cushion to direct sunlight or other sources of ultraviolet light.

DISPOSAL: When properly used and disposed, there are no known environmental hazards associated with the components of the PostureLITE cushion. Dispose of product and/or components in accordance with the applicable regulations in your jurisdiction. **DO NOT** incinerate.

PARTS DETAIL AND PRODUCT SPECIFICATIONS:


- A. Cover:** Protects cushion insert. Top and sides are Polyurethane-coated Polyester. Bottom is PVC (DEHP and BPA free).
- B. Cushion Insert:** Consists of a foam core and air expansion chamber at the front and sides. Polyurethane foam encased in PVC (DEHP and BPA free).
- C. User Manual:** Instructions on placement, care, warranty and other information about the product.

SIZING:

Item #	PostureLITE Cushion				To Fit Wheelchair Range			
	Width		Depth		Width		Depth	
	in.	cm	in.	cm	in.	cm	in.	cm
POSTUREL1	16.75	42.5	17.25	44	15 - 16	38 - 40.5	16 - 17	40.5 - 43
POSTUREL2	18.75	47.5	17.25	44	17 - 18	43 - 45.5	16 - 17	40.5 - 43
POSTUREL3	20	51	17.25	44	19 - 20	48.5 - 51	16 - 17	40.5 - 43

Average cushion height is 3 in. (7.5 cm).
 Average cushion weight is 1.4 lbs. (0.6 kg).
 Note: All measurements are approximate.


© 2011 ROHO, Inc.
 The following are trademarks and registered trademarks of ROHO, Inc.:
 ROHO® and shape fitting technology®.
 PostureLITE™ is a trademark of Comfort Concepts Pty Limited.

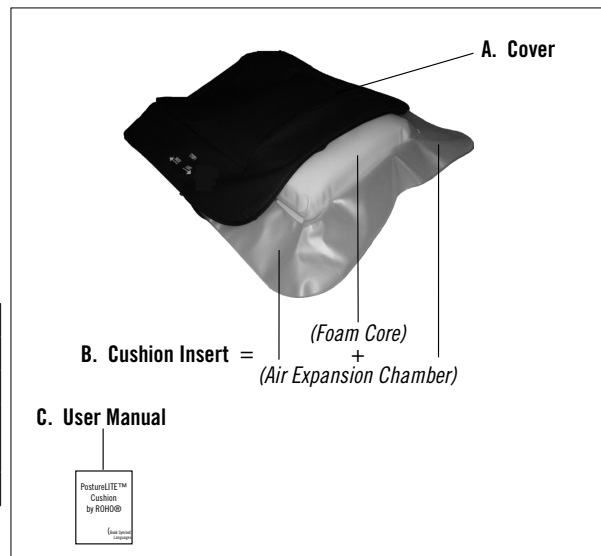

PostureLITE cushion manufactured by Comfort Concepts Pty Limited for ROHO, Inc.
 100 N. Florida Ave.
 Belleville, IL 62221-5429
 toll-free 800-851-3449 • phone 618-277-9150
 fax 618-277-6518 • www.therohogroup.com
 email: cs@therohogroup.com



POSTURELITE CUSHION PLACEMENT:

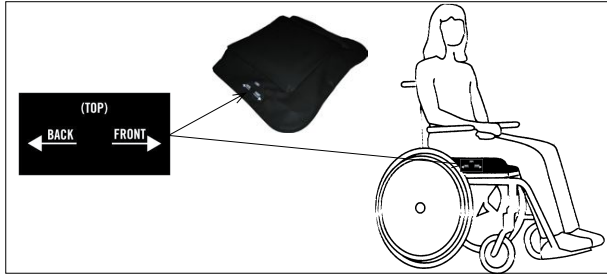


1. Use directional label on right side of the cover to properly place cushion in seat. Air expansion chamber will be at the front and sides.
2. Have user sit on the cushion. Air expansion chamber should fill with air. Check to make sure cushion is properly sized to user and wheelchair.

Note: If cushion slides on chair, make sure non-skid **BOTTOM** of the cushion cover is facing down.

REMOVING AND REPLACING COVER:

To Remove PostureLITE Cover:

1. Unzip cover.
2. Remove cushion insert.

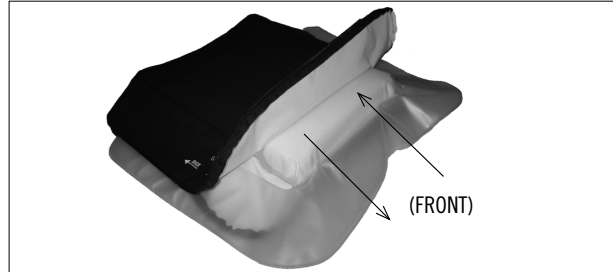

To Replace PostureLITE Cover:

1. Unzip cover.
2. Insert **BACK** of foam core into center cover pocket.
3. Lay air expansion chamber flat within outer portion of cover and close the zipper.

CLEANING AND DISINFECTING:

Cleaning and disinfecting are two separate processes. Cleaning must precede disinfection. The PostureLITE cushion must be cleaned, disinfected and checked for proper functioning between multiple patient use.

PostureLITE Cushion COVER:

Cleaning: Remove cover. Machine wash cover warm, gentle cycle. Tumble dry low.

Disinfecting: Hand wash in warm water using 1 part household liquid bleach per 9 parts water, then rinse thoroughly. Cover can also be machine washed in hot water (60°C) to disinfect. Tumble dry low.

PostureLITE Cushion INSERT:

Cleaning: Wipe using a soft, damp cloth and mild soap. Air dry.

Disinfecting: Wipe using a soft cloth or sponge and a solution of 1 part liquid household bleach per 9 parts water. Rinse with fresh water. Air dry.

Note: Follow safety guidelines on bleach container.



DO NOT:

- use cleaning products containing petroleum or organic solvents including acetone, toluene, MEK, naphtha, dry cleaning fluids, adhesive removers, etc.
- machine wash or machine dry cushion insert.
- use abrasives such as steel wool or scouring pads.
- use caustic, automatic dishwasher detergents.
- expose product to ultraviolet light or ozone gas cleaning methods.
- use oil-based lotions, lanolin, or phenolic-based disinfectants on your product, as they may compromise the integrity of the material.
- autoclave, steam clean, or expose cushion to temperatures in excess of 150°F (65°C).



LIMITED WARRANTY

WHAT IS COVERED: ROHO, Inc. covers any defects in materials or workmanship of the product.

FOR HOW LONG: 12 months from the date the product was originally purchased.

WHAT IS NOT COVERED: A product that has been misused, damaged by an accident, or damaged from “an act of Nature”, e.g., flood, tornado, earthquake, fire. Any substitutions of supplied components or modification to the product will void the warranty.

WHAT THE CUSTOMER MUST DO: **Within the U.S.A.**, the customer must contact ROHO, Inc. Warranty Service Department at 1-800-851-3449 toll free. **Outside the U.S.A.**, contact the nearest ROHO International distributor. See website for current list of ROHO International distributors (www.therohogroup.com).

QUESTIONS: Call our Warranty Service Department toll free **from U.S.A.** at 1-800-851-3449. **Outside the U.S.A.** call your ROHO International distributor. See website for current list of ROHO International distributors (www.therohogroup.com).

DISCLAIMER: Any implied warranties, including the **MERCHANTABILITY** and **FITNESS FOR A PARTICULAR PURPOSE** are also limited to the same time period of the product types referenced above in the “For how long” section. Any action for a breach of such warranties or any express warranties herein must be commenced within this time. ROHO, Inc. shall not be responsible for any incidental or consequential damages with respect to economic loss or injuries to property, whether as a result of a breach of express or implied warranties, or otherwise.

YOUR RIGHTS AND THE EFFECT OF THIS WARRANTY UPON THEM: This warranty restricts the duration of all implied warranties as noted above and limits or excludes recovery for incidental or consequential damages. Some jurisdictions do not allow limitations on how long implied warranties will last and some jurisdictions do not allow the exclusion or limitation of consequential or incidental damages, so those provisions may not apply to you. While this warranty gives you specific legal rights, you may also have other rights which vary from jurisdiction to jurisdiction.

