
Staying Home Corporation

2501 Anaconda Road Harrisonville, MO 64701 816-380-2427 www.stayinghome.com

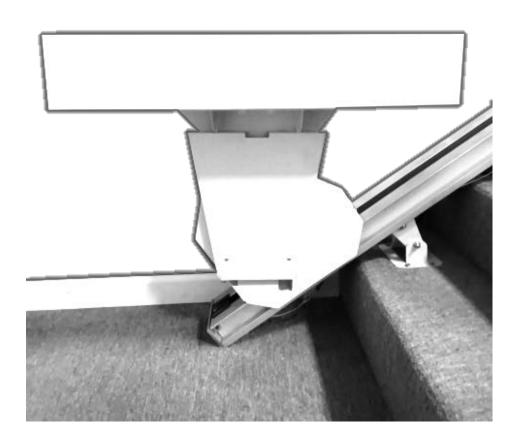

Crest Cargo Lift Installation Manual

June 25, 2019




Preliminary Checks

____ Owner's Manual

Installation Site Requirements


 115 VAC, 60 Hz, 3-wire grounded outlet within 6' of a some states may require dedicated outlet. Stair angle between 25° and 45°. Indoor installation. 	of the top of the staircase.
Included with Shipment	Tools Required
Aluminum track with gear rack and splice bars	Set of nut driver bits
(4-6) Track mounting brackets	Set of screwdrivers (phillips)
Chassis	Tape measure
(2) Plastic cams	3/8" reversible drill w/8" extension
Footrest	Allen wrenches
Cargo Lift	Portable band saw or hack saw
Battery Charger with 6' cord	(for cutting stock track) Small level
Header cover	Combination wrench (7/16" & 1/2")
Upper and lower track end caps	Tools to remove handrail if interferes
(2) Wireless call/send key fobs	with the travel of the stair lift
Small parts bag	
Installation Manual	

Underside of Chassis

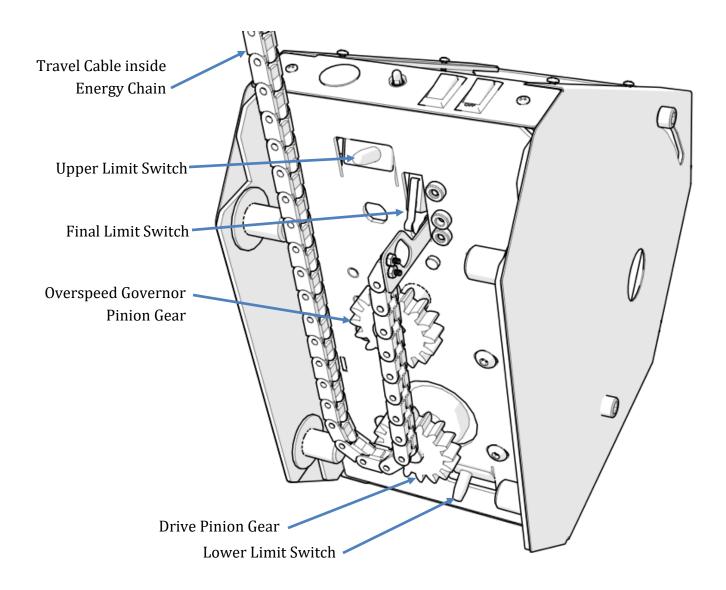

Identify Track Pieces

Red paint on the end of the gear racks indicates that end has been precision machined. Only butt-up the ends of gear racks that are red. If the end is not painted, it must be put toward the end of the track.

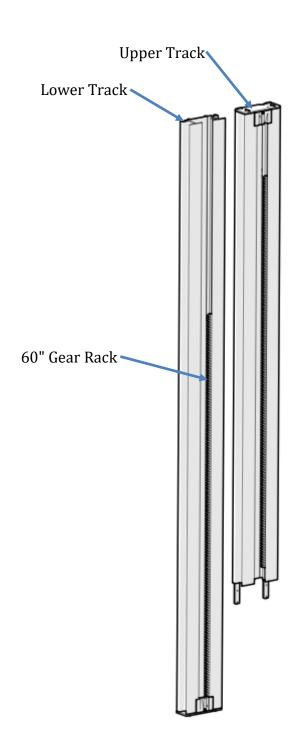

Installation Procedures for Stock Units

Track Installation: Track and rack should be cut before installing them on the staircase


Tracks are packaged in individual boxes. You will generally have 2-3 sections of track with steel gear rack, (4-6) track mounting brackets located in a small parts box located in seat box, and 1-2 sets of splice bars already pre-mounted on the track.

Note: The upper and lower ends of the track sections are marked top & bottom. *The gear rack is already inserted in the gear rack channel of the track.*

Look up the stairs and determine if the track is to be installed on the left side or right side. The gear rack should always be located on the right side of the track regardless of which side of the stairs the track is mounted.

Measure the distance from the top nose of the staircase to the bottom floor and add 7" (note: be sure to verify clearance from top nose to any obstruction at the top such as a door or door frame that would cause the chair to hit or block the seat from swiveling.) It may be necessary to ramp the track away from the obstruction, consult factory for assistance if needed.

Cut the track at the top, leaving the factory cuts at all other sections. Cut the gear rack flush with the top of track.

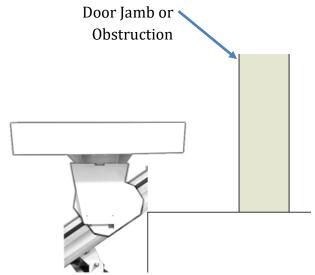

Splice Track Pieces Together

Check the edges of the outside C-channels for rough edges. If there are any burrs, file them off before splicing the track


Position the splice bars so they are half way inserted into one track.

Tighten the set screws in the bars.

Tap the large end of the 3/16" pin half way into one of the tracks.


Slide the other track onto the splice ensuring that the track pieces align on all sides.

Tighten the remaining set screws to lock everything together. Run your finger over the outer edges of the tracks to assure there are no significant edges protruding. If there are, separate the track slightly and realign as required to obtain a smooth transition between the track channels.

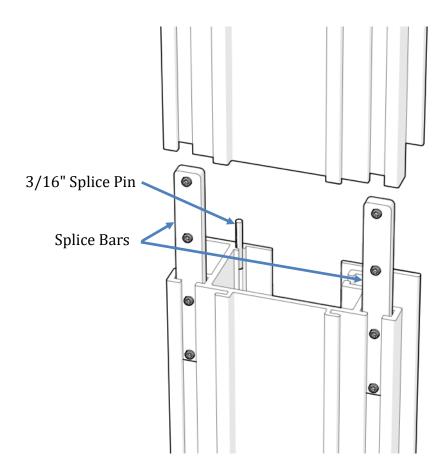

Secure the Track to the Stair Treads


When the track is installed, there should be a 1" gap between the underside of the track and the stair nosings and 1/4" to 1/2" off the lower landing floor. The track should be at least 4" away from the wall or other obstruction.

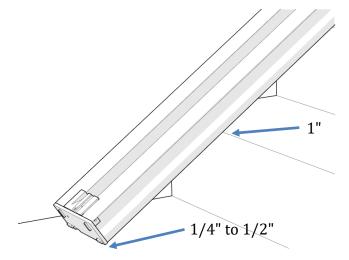


Install rail brackets by loosening the screws and snapping each bracket edge into the slot or slide the brackets on from the top of the rail. The rail brackets are designed to lean the track toward the wall to offset the weight of the unit and user, therefore, the nuts on brackets should be on the staircase side (toward the middle of the staircase).

Place one bracket on first tread, one on the top tread, and one bracket immediately above and below each splice. Then use the other brackets to support any section of track left unsupported and measuring greater than 48".

Fasten each bracket to the treads with (4) #14 X 2" hex head lag screws.

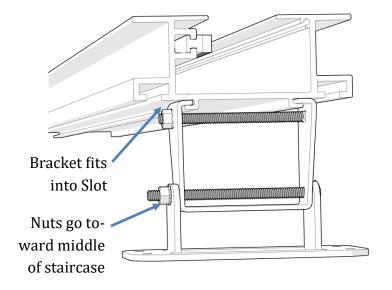

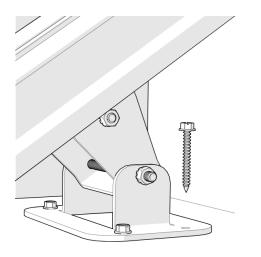

Insert the Chassis into the Track


Remove the upper most section of gear rack to allow you to slide the chassis into the track easily.

Ensure the glide blocks and shim spacers are properly installed on the chassis axles. **One shim goes onto** each axle on the gear rack side only. The opposite side does not need shims.

Failure to have the proper shims in place will cause the pinion gear not to engage in the gear rack properly.

The chassis can now be inserted into the top end of the track.

Feed the travel cable/energy chain inside the track. Make sure the energy chain does not snap apart.

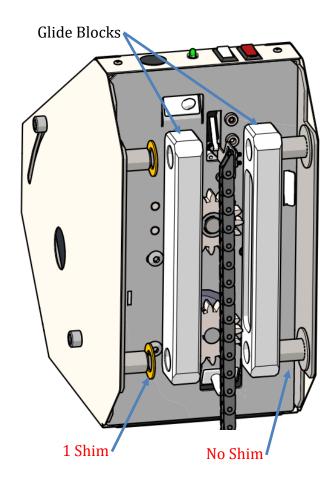


Flip the red RESET/OFF switch to the RESET position. The lift will beep and is now ready to be run down onto the gear rack using the black INSTALLATION SWITCH on top of the chassis.

Run the chassis down the track to the lower limit.


Replace the top piece of gear rack in the track.

With the chassis parked at the bottom limit, slide the energy chain/travel cable down along the inside of the track, beneath the gear rack, until there is a small loop under the chassis, beyond the point of where the travel cable is attached to the chassis.

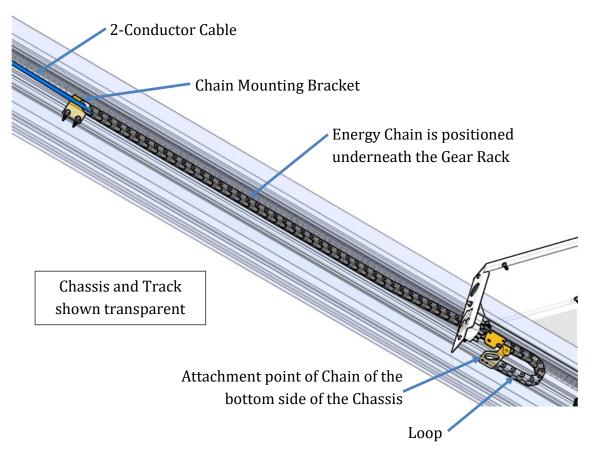

Adjust position of travel cable so there is a small loop in the Chain.

With the chain in position, fasten the chain mounting bracket to the track with (2) self -drilling screws.

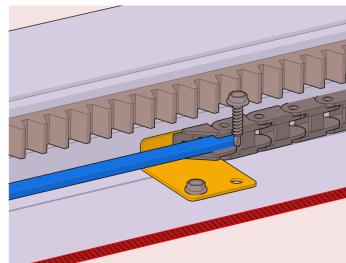


Place the upper limit cam inside the track and secure by tightening the phillips head screw.

Place the end plate onto the end of the track while routing the travel cable through the slot in the bottom of the end plate. Tighten all (6) set screws in the end plate.


Tighten the gear rack compression screw.

Connect the travel cable to the charger.

Plug the charger into a 120V outlet.

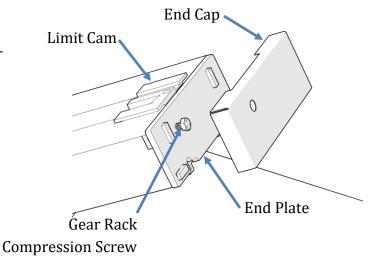


Loosen the (4) socket head screws on the sides of the chassis (2 on each side).

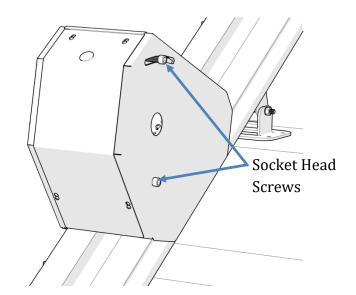

Place the footrest assembly onto the two socket head screws on the chassis. Use the keyhole pattern that offsets the footrest assembly toward the top end of the chassis.

Connect the wire harnesses between the chassis and footrest assembly.

Level the footrest and tighten the (4) socket head screws (2 in front and 2 on back). *Note that the ETL* listing is valid only when the track installed at maximum angle of 45°.

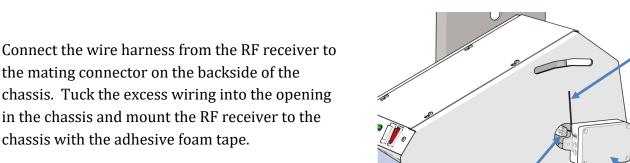


in the chassis and mount the RF receiver to the chassis with the adhesive foam tape.

Adjust the position of the antennae as necessary.

The swivel stop is factory set for a left-hand unit. If the lift is installed on the right-hand side of the staircase, move the seat swivel stop screw to the opposite hole on top of the footrest assembly.

Place the plastic thrust washer onto the swivel pin.

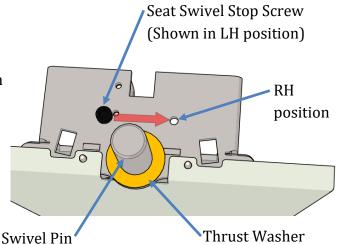

Wire Harness

Footrest

Assembly

Wire

Harness

Antennae

RF

Receiver

RF Remote Call-Send Controls

The two key fobs (transmitters) have been factory programmed.


If you are installing two lifts in the same home, it is not necessary to change the program. However, two lifts cannot be operated from the remote fobs at the same time. Each transmitter and receiver is unique and has its own rolling code.

If a key fob needs to be programmed:

- 1. Press the program button on the RF receiver 5 times until the letter #\(\text{R}\) is displayed. There should now be a slow flashing dot on the display.
- 2. Press and hold either button on one of the transmitters and then release when the dot flashes faster.
- 3. Repeat step 2 for each additional transmitter.

After the last transmitter is programmed, the LED display



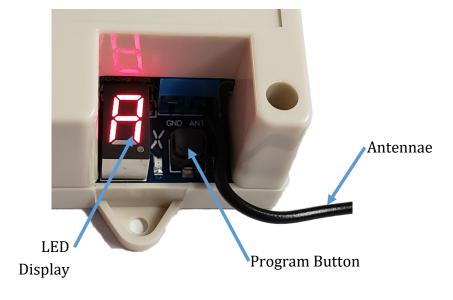



Key Fob Transmitter #74034

AUDIO/VISUAL STATUS INDICATIONS

The lift is equipped with a chassis mounted 3-color LED display light to indicate the operating condition of the lift. Accompanying this is an audible alarm that will sound accordingly.

STATUS LIGHT	AUDIBLE ALARM	CONDITION
RED-YELLOW-GREEN	1 Second beep	Power-up Cycle
SOLID GREEN	None	Ready to Operate
YELLOW BLINK	1/4 Second beep (30 Seconds)	Batteries are not being charged (Beeping repeats every 10 minutes)
YELLOW PULSE	1/2 Second beep (5 Minutes)	Low Battery Voltage (under 22.5 volts)
2 YELLOW FLASHES	1/2 Second beep	Seat Swivel Latch Sensor
3 YELLOW FLASHES	1/2 Second beep	Footrest Obstruction Sensor
2 RED FLASHES*	3 Second beep	Sensor Fault (2 or more sensors engaged)
3 RED FLASHES*	3 Second beep	Final Limit
4 RED FLASHES*	3 Second beep	Travel Timeout (Motor running for more than 2-1/2 minutes)
YELLOW SOLID (30 SECONDS)	1/2 Second beep (30 Seconds)	Overload (Over 22 amps for 3 seconds)
RED SOLID (3 SECONDS)	3 Second beep	Key turned off while traveling(Optional)
OFF (SLEEP)*	None	Very Low Battery Voltage (Under 20.5 Volts for 5 minutes)
FLASH = 2 per second		
BLINK = 1 per second		
PULSE = 1 per 10 seconds		
*Requires power to be cycled OFF/ON.		

Completion Procedures

SERVICE LINE 877.378.4275 (Mon-Fri 8:00 – 5:00 CST)

Completion Checklist

The following features must be verified as operational before the stair lift can be released for use:	
Upper and lower limits: Verify the lift stops automatically at the top and bottom of the track.	
Charger: Verify that the light on the charger changes from green to amber when the lift is running.	
Running Clearance: Verify the lift clears all obstructions.	
Unit Control: Verify control functions in both directions.	
Call/Send Remotes: Verify both call/send controls operate the lift up and down in the appropriate direction.	
Track:	
Important: Top Track End Plate: It is imperative the 6 small set screws under the top track end plate are very to	ight.
Important: Lower Track End Plate: It is imperative the 6 small set screws under the lower track end plate are v tight.	ery
Track Mounting Brackets: Verify all track mounting brackets are securely attached to the stair treads, the track and at the pivot points.	
Track Splice: Verify track joints are smooth and free of burrs and gaps.	
Verify the rack tension screw is tight.	
Verify chassis gear and rack is tight and no excess play.	
Verify the gear rack is lubricated with lithium or other all-purpose grease (tube of lubricant is provided)	
Verify the top track cover is securely fastened to the track.	
Verify the inside of the track is free of foreign objects.	
Footrest:	
Verify the footrest is level.	
Verify the footrest clears all stair nose.	
Verify the lift stops when the footrest runs into an obstruction in the up direction.	
Verify the lift will run down while the footrest is obstructed.	
Verify the footrest folds up and stays in the up position.	
Seat:	
Verify the seat is level and securely fastened.	
Verify seatbelt & buckle are secure and function.	
Verify the seat swivels toward the upper landing and locks into position.	
Verify the lift will not operate when the seat is not locked in the riding position.	
Verify the seat will fold up and stay in the up position.	
Verify both arms will fold up and stay in the up position.	
Clean Up: Verify the stair lift has all grease, dirt, etc. cleaned off.	
Before leaving the jobsite:	
Clean up work area.	
Assist customer on how to use the unit and ride it properly.	
Demonstrate proper operation, seatbelt use, lubrication and maintenance procedures to the user of the lift and other family members at the residence.	
Give customer a copy of the Owner's Manual for reference. Please take time to insert your business card or write your company name, address and telephone number inside the Owner's Manual in the area provided.	

